

User Manual

oscident IPR Strips · Openers · Polishers

Intende use

Bad Homburg, 02.01.2015

A Strip/Opener/Polisher coated with polishing particles. It is used for correction of the teeth surface or for restauration.

Preparation

1. Carefully check all product surface if there are any deformation, pin holes and foreign substance
2. Sterilization (Autoclave: Sterilization at 132°C for 15min., Dry 30min.)
 - 2.1. Follow the manufacturer's sterilization method and operating instructions
 - 2.2. Place all instruments avoid contact each product surface by using sterilization wrap

Method of use

1. Follow the hospital manual for dental abrasive strips
2. Lace the strip between the teeth and gently move backward and forward to polishing, cutting and deletion of the teeth surface
3. Use different one if the product having been dropped, corrosion or cracks found during use

Storage and keeping conditions

1. Cleaning all part of the instrument using cleaning solution or waters to remove human blood or foreign materials. Completely open the instrument hving overlapping area before start the cleaning steps.

Warning an precautions

1. Do not use the product other than the intended purpose
2. Do not let the excessive force to product
3. Carefully read package labels to ensure use of the appropriate device
4. Eye protection must be worn to guard against ejected particles
5. Avoid using when polishing surface are broken or blunted
6. Care should be taken to avoid injury to patient or user caused by sharp edge
7. Care should be taken to avoid damage he adjacent teeth